Oracle® for Cats and Dogs Blood Glucose Test Strips

Ver. 1.0 2025/09
312-4257300-015

⚠ Warnings and Precautions

Product code: GMP-A1S50

- ▶ **For Veterinary Use Only.**
- ▶ For Cats and Dogs use only.
- ▶ For single use only.
- ▶ The recommended control solution range is on the test strip vial for dogs and cats.
- ▶ Please read this sheet and your meter Owner's Manual before you use this test strip.
- ▶ Use only **Oracle® for Cats and Dogs** Blood Glucose Test Strips with **Oracle® for Cats and Dogs** Blood Glucose and β -Ketone Monitoring System (GMP-A1) to obtain accurate results, and be covered by the manufacturer's warranty.
- ▶ Severe dehydration and excessive water loss may cause false low results. If severe dehydration is suspected in your pet, consult a veterinarian / veterinary surgeon immediately.
- ▶ Keep test strips and lancets away from small children. If swallowed, consult a doctor immediately for advice.
- ▶ The used test strips are potentially biohazardous. Please dispose of them carefully according to your local regulations.
- ▶ This system does not serve as a cure for any symptoms or diseases. The data measured is for reference only. Always consult a veterinarian / veterinary surgeon to have the results interpreted.

Intended Purpose

Oracle® for Cats and Dogs Blood Glucose Test Strips, when used together with the **Oracle® for Cats and Dogs** Blood Glucose and β -Ketone Monitoring System (GMP-A1), allows cat and dogs' blood glucose levels to be measured by their owner for self-testing and by veterinarian professionals in clinical settings as an aid to monitoring the effectiveness of diabetes control in the animal. It is intended to be used for the quantitative measurement of blood glucose in fresh whole blood samples. It should not be used for the diagnosis or screening of diabetes mellitus.

Professionals may test with capillary and venous blood sample from the animal's paw pad and marginal ear. Self-testing is limited to capillary whole blood testing.

Test Principle

The test strips draw a blood sample by capillary action and use electrochemical methods to generate an electric current from the reaction between glucose and enzymes. The meter then detects and translates this current into the observable reading.

Limitations

- ▶ Use only heparin for anticoagulation of fresh capillary or venous whole blood.
- ▶ Hematocrit: The hematocrit level is limited to between 0% to 70%.
- ▶ Altitude Effects: Altitudes up to 10,742 ft (3,275m) do not affect test results.


Storage and Handling

- ▶ Refer to the expiry date on the test strip package. Do not use the test strips if they have expired.
- ▶ Refer to the expiry date on the test strip vial after first opening for vial-packed strips.
- ▶ Store the test strips at 2°C to 30°C (35.6°F to 86°F) and 10% to 90% relative humidity.
- ▶ Store the test strips in their original vial only. Do not transfer them to a new vial or any other containers.
- ▶ Use each test strip immediately after taking it out of the vial or individual foil packet.
- ▶ Close the vial immediately after taking out a strip. Keep the strip vial closed at all times.
- ▶ Keep the test strips away from direct sunlight. Do not store the test strips in high humidity.
- ▶ Do not touch the test strips with wet hands.
- ▶ Do not bend, cut, or alter the test strips.

Performing a Blood Test

Please refer to the meter Owner's Manual for detailed operation instructions.

1. Clean and dry your hands.
2. Prepare lancing device.
3. Insert the test strip into the meter.
4. Prick the puncture site.
5. Obtain and apply the blood sample.
6. Remove the used test strip and lancet.

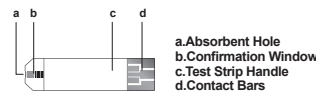Reading Your Result

The blood glucose readings deliver plasma-equivalent results and are displayed either in milligrams of glucose per deciliter of blood (mg/dL) or in millimoles of glucose per liter of blood (mmol/L). The measurement range of this meter is 10 to 800 mg/dL (0.56 to 44.4 mmol/L).

Please consult a veterinarian / veterinary surgeon to determine the target glucose values for each animal.

If the test results are unusual or inconsistent:

- ▶ Make sure the confirmation window of the test strip is completely filled with blood.
- ▶ Check the expiry date of the test strips.
- ▶ Check the performance of your meter and test strips with the control solutions.

NOTE:

Low or high blood glucose readings can indicate a potentially serious medical condition. If the blood glucose reading is unusually low or high, repeat the test with a new test strip. If the reading is not consistent with any visible symptoms, contact a veterinarian / veterinary surgeon and follow his or her treatment advice.

Chemical Components

- Glucose dehydrogenase (FAD dependent, microorganism) 8%
- Electron shuttle 55%
- Enzyme protector 8%
- Non-reactive ingredients 29%

Additional Information for Veterinary Professionals

Veterinary professionals may use test strips to test capillary and venous whole blood.

Use fresh whole blood samples only.

- Sample Size: 0.5 μ L
- Reaction Time: 5 seconds
- System Measurement Range: 10 to 800 mg/dL (0.56 to 44.4 mmol/L)
- Hematocrit Range: 0% to 70%

Symbol Information

Symbol	Referent	Symbol	Referent	Symbol	Referent
	Use-by date		Batch code		Manufacturer
	Temperature limit		Caution		Do not re-use
	Humidity limitation		Consult instructions for use or consult electronic instructions for use		

 **Tremblay Harrison International Inc.**
1684 Dufferin Street
Toronto, ON, M6H 3M1
1-866-829-7926
oracled diabetes.com

For Veterinary Use Only.
For cats and dogs Use Only.

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